GEHOOR in die Moot

Dr René Hornby Oudioloë/Audiologists

Newsletter/Nuusbrief

No 1 of/van 2018

Email/E-pos: admin@gehoorindiemoot.co.za

Appointments/Afsprake: (012) 331 4469

Fax/Faks: (012) 330 1387

Website/Webtuiste: www.rhearing.co.za

PRAKTYKNUUS / PRACTICE NEWS

Oktober 2017

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PROMOSIE: SLEGS R35 PER PAKKIE SINK-LUG BATTERYE!

PROMOTION: ONLY R35 PER PACK OF ZINC-AIR BATTERIES!

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Humidity and temperature also influence battery life. Zinc-air batteries require air to work properly, so high levels of humidity can negatively influence their functioning. Storing your hearing aids in a drying kit at night may help to reduce the influence of moisture. Batteries also tend to work best at room temperature, so it is not a good idea to store them in places with extreme temperature variations, e.g. your car. Contact with other **metals** can cause batteries to lose power, even if the sticker has not yet been removed. It is therefore best to store batteries in their original packaging and not in your pocket or purse.

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HEARING AID WARRANTY

Most hearing aids carry a 2 or 3 year warranty. Please keep the following in mind:

- Should your hearing aid have any electronic faults, the faulty parts will be replaced free of charge during this period.
- Your audiologist recommends annual servicing of your hearing aid in order to guarantee reliable functioning. A small amount of sweat or dust may negatively influence the functioning of your hearing aid's microphones. Should you perspire a lot, it is recommended that your hearing aid be serviced every 6 months.
- When your hearing aid is serviced by the manufacturer during the warranty period, the service will be free of charge. A small insurance fee is charged by our practice.
- When your hearing aid is in need of a repair or service, please bring it to the practice and the audiologist will send it to the manufacturer.
- It will take about 5 working days for your hearing aid to be repaired/serviced. You will receive a call from the practice when it can be collected.
- When your hearing aid has been serviced, it is advisable to make an appointment for a follow-up hearing test so that your hearing aid can be programmed according to your current results.
- Servicing of hearing aids outside the warranty period is charged at a rate dependent on the manufacturer. You may request a quotation.
- If you have had your hearing aid for longer than 4 years, it is nearing the end of its lifespan. We advise against expensive repairs in such cases, and instead recommend a new device which carries a warranty.