



GEHOOR IN DIE MOOT

Nuusbrief 1
Newsletter 1
2022



Appointments/Afsprake: (012) 331 4469 / admin@gehoorindiemoot.co.za

Hearing Aids alone not enough?

There are numerous options for you if you need more than just your hearing aids to make everyday communication and listening easier.

****TV streaming:** Tired of having the TV volume on high? TV streamers send the sound from the TV directly and wirelessly to your hearing aids. You can adjust the volume through the hearing aid app.

****Partner mic:** Your partner clips on a microphone, and his voice streams directly and wirelessly to your hearing aids. Great for car conversations, noisy restaurants, conferences and school or university classes.

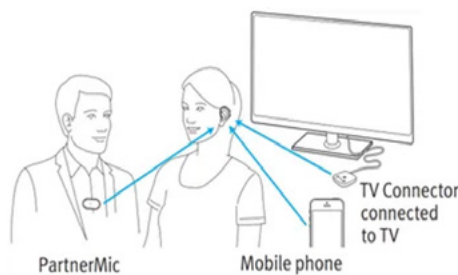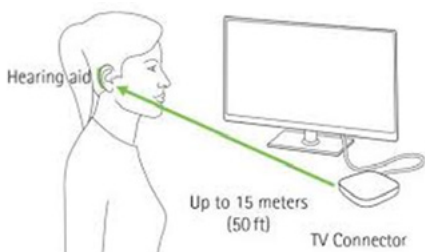

****Smartphone call streaming:** Most hearing aids can stream phone calls directly to an iPhone. There are some new hearing aids that can stream phone calls directly to Android phones, and Unitron hearing aids use Bluetooth to stream to most Android and iPhones.

AUTUMN AND WINTER SPECIALS



- * **FREE TV connector** worth R4000 with any pair of Unitron Blu non-rechargeable hearing aids
- * **FREE charging case** worth R3200 with any pair of Unitron Blu rechargeable hearing aids

WIDEX MOMENT™

THIS SOUND CHANGES EVERYTHING

- * **FREE TV Play** worth R4000 with any pair of Direct streaming Widex Moment hearing aids
- * **FREE Perfect Dry Lux** worth R2000 with any pair of non-streaming Widex Moment hearing aids
- * **FREE Charge and Clean charger** worth R4000 with a pair of rechargeable Widex Moment mRICD

Rechargeable vs Disposable Hearing Aid Battery Technology

Rechargeable hearing aids have become prolific over the past 5 years, and rechargeable battery technology is constantly improving to provide longer battery life with shorter charging periods. When making the decision to purchase rechargeable technology, it is important to keep the following in mind:



Rechargeable Technology	Disposable Technology
Uses built-in lithium-ion batteries.	Uses disposable zinc-air “button” batteries.
More expensive than disposable technology.	Less expensive than rechargeable technology.
No need to remove batteries - the entire hearing aid is docked in a charger overnight.	Batteries need to be replaced regularly and battery doors need to be opened every night.
Can last up to 30 hours on a single charge (fewer hours if streaming audio). Should be charged nightly, much like a cell phone.	Batteries need to be replaced approximately every 7 days, depending on battery size and hearing aid use.
As lithium-ion cells age, a single charge lasts for less time, and charging units usually need to be replaced after a few years.	As batteries are replaced weekly, there is no need to worry about aging batteries requiring replacement after years of use.
Most hearing aids have a standard warranty of 2 or 3 years. <i>However</i> , hearing aid chargers are considered accessories and thus only carry a 1 year warranty.	Most hearing aids have a standard warranty of 2 or 3 years.
Hearing aids are easier to handle for those with poor dexterity or eyesight, as battery doors do not need to be opened.	Hearing aids are more difficult to handle for those with poor dexterity or eyesight, due to small batteries and battery doors.
Hearing aids need to be sent to the manufacturer if batteries are faulty.	Batteries can easily be replaced at home.
Hearing aids require a model-specific charger in order to charge, and interruptions to a charging cycle (e.g. due to loadshedding) can lead to hearing aids not lasting for a full day.	Hearing aids require a specific disposable battery size (10, 312, 13 or 675). Battery stock should always be kept on hand.



‘n Paar belangrike feite om te onthou

- » Jou gehoorapparaat se werking en batterylewe word beïnvloed deur enige oormatige vog, sweet, oorwas en velolie.
- » Jou gehoorvermoë kan dalk deur die jaar afneem, en jou apparaat is dan nie meer hard genoeg gestel nie.
- » Wasproppe in oorkanale kan gehoorapparaatklank heeltemal blokkeer. Dit is ‘n algemene verskynsel en kan maklik verwyder word.